



# LiquiBand® Optima

## The fast, precise and controlled topical skin closure.

Our winged applicator allows for **effortless activation** and **controlled application**


10 SECOND SET TIME	MICROBIAL BARRIER	PRECISION APPLICATOR	NO STING	AMBIENT STORAGE	MADE IN THE UK

### ✓ UNIQUE APPLICATOR

- ▶ Winged applicator allows easy and safe priming
- ▶ Visidome reservoir provides visualization of remaining adhesive

### ✓ EFFECTIVE SCALP WOUND CLOSURE

- ▶ Safe, effective and rapid wound closure for scalp lacerations
- ▶ Precision tip ideal for areas of dense hair growth

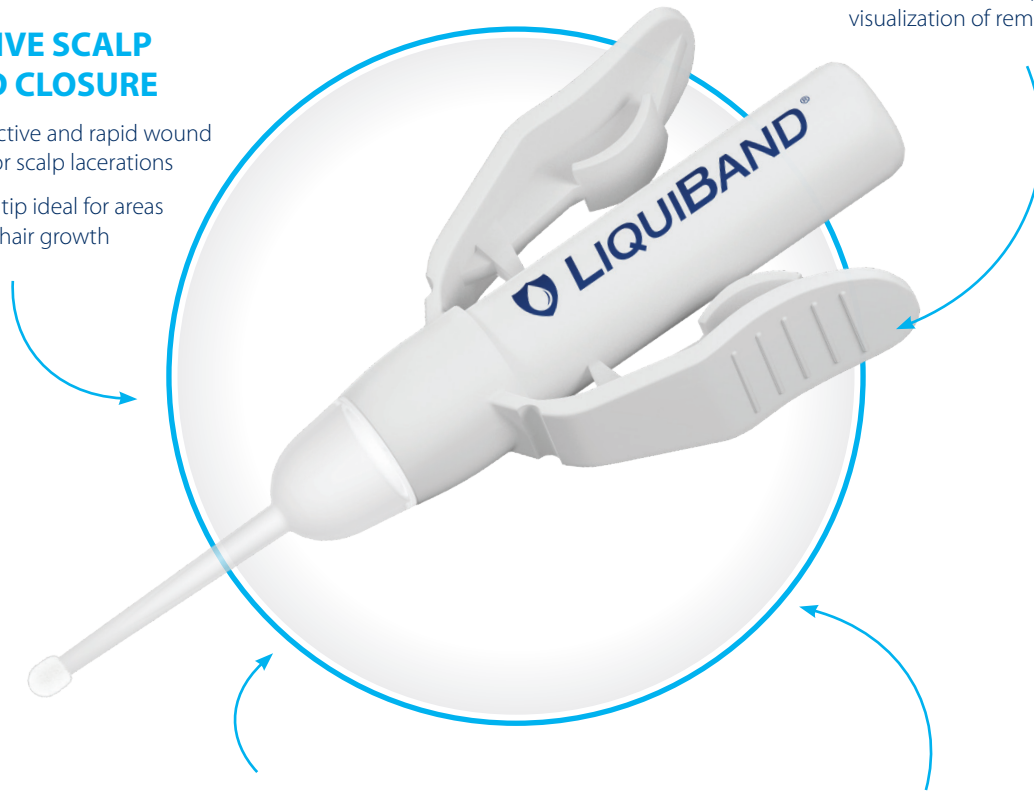


### ✓ ENHANCED PATIENT BENEFITS<sup>1</sup>

- ▶ Offers excellent post-operative cosmesis
- ▶ Provides low exothermic reaction

### ✓ BUTYL-CYANOACRYLATE BASED FORMULA

- ▶ Strong and secure wound closure
- ▶ Quick 10-second dry time
- ▶ Controlled placement of adhesive

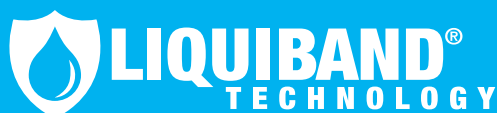

Exclusively distributed by:



Tel. No. 635-9747; 637-9464  
Fax No. 635-9748  
email: sales@lifelink-inc.com

Better wound closure starts with a better tool

## Application Guide **LiquiBand® Optima**

LiquiBand® Optima Topical Skin Adhesive has a unique, winged applicator to facilitate safe and easy activation, along with accuracy and control.

### ✔ STEP ONE

- ▶ Ensure that wound is thoroughly clean and dry prior to closure.
- ▶ While holding upright, squeeze the applicator wings until a cracking sound is heard.



### ✔ STEP TWO

- ▶ Invert the tip and gently squeeze wings to prime the device. When the tip fills with adhesive, LiquiBand® Optima is ready to use.



### ✔ STEP THREE


- ▶ While holding the wound edges together, apply an even coat of LiquiBand® Optima over the length of the wound.
- ▶ Continue to hold the wound edges together for 10 seconds.



### LiquiBand® Optima

Size	Qty Per Box	Product Code	NHS SC Code
0.5g	10	OPT001	FVF 009

\* If being used in areas of dense natural hair, take care when applying as hair follicle damage may occur if over applied.

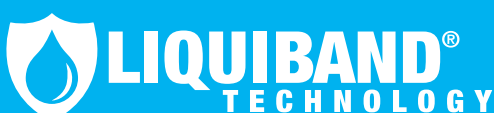

Exclusively distributed by:



Tel. No. 635-9747; 637-9464  
Fax No. 635-9748  
email: sales@lifeline-inc.com

T +44 (0) 1752 209 955  
E info@liquiband.com  
W liquiband.com